

S.K.I.P Harness **neonatal transport system**

Secure Kids In Properly

Cleaning Instructions



manufactured in New Zealand©
www.skipharness.com

Cleaning Instructions

Cleaning is best done when the dirt is still new, hence the harness should be cleaned as soon as possible after use.

The Leather of the SKIP Harness has a non-absorptive barrier on its surface stopping particles from being absorbed, still allowing at the same time for the leather to breathe. However the stitching is porous so drenching the S.K.I.P. Harness is not recommended or soaking the S.K.I.P Harness.

Cleaning of Harness

Use a small amount of warm soapy water or a light spray of 'Virkon' on the harness and wipe down with a cloth. Alternatively spray the cloth and wipe the harness, dry with a dry cloth.

Other products that can be used

- Accel prevention wipes
- PDI super Sani-cloths

Cleaning of Velcro tapes

Ensure that the Velcro tapes are cleaned regularly of any fluff from blankets or other fabrics that, with general use may become attached. This may potentially decrease the adhesiveness of the Velcro overtime if there was a large build-up of fluff.

MRSA Cleaning

Use 0.1% sodium hyperchloride solution. Wipe harness with the solution. Then leave the harness for one hour. Then wipe down again with sunlight liquid and dry. This practice is in line with infection control policies.